

Planmeca PlanScan® – just plug and play


*Planmeca is proud to introduce **Planmeca PlanScan®** – the world's first dental unit integrated intraoral scanner. This unique integration guarantees a smooth workflow, as real-time scanning data is immediately available from the chairside tablet.*

World's
first dental
unit integrated
intraoral
scanner



Smooth and effortless workflow

- Dental units adapt at once from scanning to other treatment needs
- Use the same foot control as with other treatments
 - Toggle between prep, opposing and buccal views
- Intuitive scanning with optional sound guidance from the tablet
- Quick scanning, data is instantly available for design

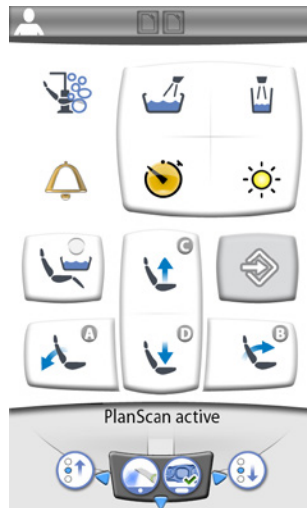

The same intuitive user interface as with other treatments.

Intuitive scanning with optional sound guidance from the tablet.



Easy sharing between dental units

- The plug-and-play scanner is used like any other instrument
- The tablet can be freely moved according to user preference
- Also available for retrofitting



Plug & Play

Ideal for small treatment rooms

There is no need for separate carts as everything is integrated.

Excellent working ergonomics

- Constant access to real-time scanning data
- All data can effortlessly be seen from the tablet, with minimized body movements
- Disable scanning quickly to continue other treatments with just a press of a button
- Exceptional for two-handed dentistry – operators working alone have perfect access to the scanner, suction tubes and tablet




Ordering information

Factory installation / retrofit

- **Planmeca Compact™ i Touch** (v1/v2):
FE003800 Planmeca PlanScan interface
- **Planmeca Sovereign® Classic:**
FE003801 Planmeca PlanScan interface

Planmeca PlanScan® interface includes

- Planmeca PlanScan holder ●
- Tablet holder ●
- Firewire cabling and 4.5m firewire extension cable ●
 - For transferring the Planmeca PlanScan image from the scanner to a PC
- Ethernet cabling and 9m Ethernet extension cable ●
 - For the foot control and **Planmeca Romexis®** connection
 - For Planmeca PlanScan activation by the left side button
 - For choosing the scanning indication (preop, prep, opp, bite)
 - To start scanning, end scanning and create a model
 - For adjusting the field of view during scanning
- Power (same as for the medical grade monitor) ●
- Port for charging the tablet ●
- Firewire port for Planmeca PlanScan ●
- Latest dental unit software* ●
- Planmeca Romexis and the dental unit user manual in a USB memory stick (all language versions), the Planmeca PlanScan interface and tablet holder installation manuals as a paper version (in English).

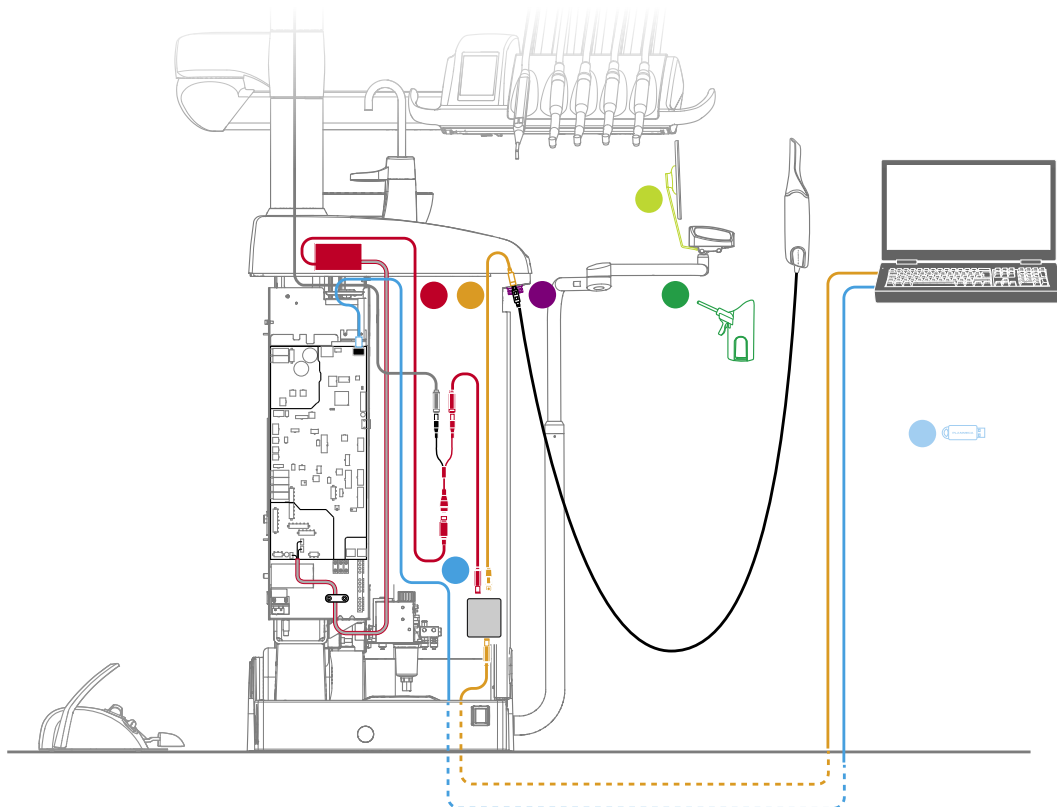

*Compatible with Planmeca Sovereign Classic 1.2.0 software / Planmeca Compact i v2 software 7.4.10 / v1 software 6.4.10

Options

- FE003818 10 m firewire extension cable

Requirements

- **Planmeca PlanScan**® intraoral scanner
- Tablet e.g. Sony Xperia Z2, a full HD input tablet is required for proper scaling of **Planmeca Romexis**®
- Planmeca Romexis software version 3.8. or later
- Suction arm with a **Flexy**™ holder
- Foot control with side buttons
- W-lan router – it is highly recommended to use a router operating at 5 GHz
- Splashtop remote connection application (download from Google Store)

N.B. Planmeca Sovereign Classic: An assistant Panasonic intraoral camera cannot be used at the same time as Planmeca PlanScan

Brochure

- Planmeca PlanScan – the ultimate scanning experience
 - Browse, download or send the brochure by email
<http://brochurekit.planmeca.com>
 - Download the pdf version (English)
http://publications.planmeca.com/Brochures/CAD_CAM/PlanScan_integrated_en_low.pdf
 - Order the paper copy
<http://materialbank.planmeca.com>

Order forms

Can be downloaded from Dealer Support

Pricing

Consult your area export manager

Availability

Available for exhibition purposes now.

Consult your area export managers about an exhibition kit

Available for orders from 8 December 2014.

For further information, please contact

Ms Kaisu Ilomäki
Product Manager, Dental Care Units division
Tel. +358 20 7795 433
kaisu.ilomaki@planmeca.com